

# PARTIAL M-FINGER

## DELIVERY INSTRUCTIONS

### INLCUED IN PACKAGE

- Definitive Device
- Spare Parts

### FITTING INSTRUCTIONS

- Don device on patient.
- Check fit and alignment.

### IF BOA ATTACHMENT IS USED

- Adjust tension and placement of BOA system for optimal patient use.
- Attach BOA to pre-preg cuff with a rivet.  
See reference materials for further details.

### CARE & MAINTENANCE

- New users should start with a wearing schedule and build up to a full day of wear.
- Patients should clean the silicone liner with soap and water or hand sanitizer daily.
- Do not expose the device to corrosive or caustic materials, UV rays (such as tanning beds) or temperature extremes.  
See reference materials for assembly/disassembly instructions.\*\*

### CONTACT INFORMATION

Hanger Fabrication Network  
Attn: Silicone Interfacing Department  
10 County Line Drive, Unit E  
Cromwell CT 064416  
Phone: 860-667-5357 Fax: 860-632-0013