

M-THUMB

CHECK FIT INSTRUCTIONS

INLCUDED IN PACKAGE

- M-Thumb, hardware and separate mounting base.
- Custom designed silicone and carbon frame

FITTING INSTRUCTIONS

- Don silicone and carbon frame to assess fit.
- If the liner and socket fits well, proceed to alignment and mounting.
- If socket does NOT fit, please mark all adjustments necessary and return to the address below for new fabrication. **PLEASE CONTACT US BEFORE PROCEEDING.**

MOUNTING

- Mark proper alignment of the thumb.
- Apply a minimal amount of 60 second epoxy to tseparate white mounting base, NOT THUMB BASE. Hold in place until set.
- Once set, please return to the address below for final fabrication.
- Please include any additional notes necessary.

RETURN MATERIALS FOR FABRICATION TO:

Hanger Fabrication Network
Attn: Silicone Interfacing Department
10 County Line Drive, Unit E
Cromwell CT 064416
Phone: 860-667-5357 Fax: 860-632-0013