

# TITAN THUMB

## DELIVERY INSTRUCTIONS

### INLCUDED IN PACKAGE

- Definitive Device
- Spare Parts

### FITTING INSTRUCTIONS

- Don device on patient.
- Check fit and alignment.
- Adjust tension of allen screw on thumb to tighten or loosen rotation and flexion tension
- See reference materials for further details.

### CARE & MAINTENANCE

- New users should start with a wearing schedule and build up to a full day of wear.
- Patients should clean the silicone liner with soap and water or hand sanitizer daily.
- Do not expose the device to corrosive or caustic materials, UV rays (such as tanning beds) or temperature extremes.  
See vendor reference materials for detailed assembly/disassembly instructions.

### CONTACT INFORMATION

Hanger Fabrication Network  
Attn: Partial Hand Department  
10 County Line Drive, Unit E  
Cromwell CT 064416

Phone: 860-667-5357  
Fax: 860-632-0013